Intensiv HygienicTray HT1000-HT2000-HT3000

DE

Produktbeschreibung

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GB

Product description

- Fully dismantable in two parts.
- Suitable for the standard sterilization tray.
- Stainless steel.

HT1000

- Ideal arrangement of the 13 FG and 5 RA perforations for easy access to the instruments.
- Dimensions:
H x L x W: 30 x 50 x 40 mm

HT2000

- Ideal arrangement of the 26 FG and 10 RA perforations for easy access to the instruments.
- Dimensions:
H x L x W: 30 x 104 x 44 mm

HT3000

- Ideal arrangement of the 36 FG and 18 RA perforations for easy access to the instruments.
- Dimensions:
H x L x W: 30 x 150 x 44 mm

Practical recommendations

- Arrangement of Intensiv Rotating Instruments for daily use in dentistry protocols.
- Storage of Intensiv Rotating Instruments.

Instructions for use

- Application by dental professional only.
- To open the tray, pull up the cover while holding the base.
- When opening the tray, keep in a horizontal position to avoid the instruments falling out.
- Close the tray by pressing down the cover until a "click" is heard.
- If the cover is too loose, remove it and press it lightly on the ends, in order to reach sufficient pressure.

Maintenance and Sterilization

- To facilitate cleaning, dismantle the Intensiv HygienicTray completely (cover, instrument tray).
- The HygienicTray is supplied nonsterile. Prior to first use and immediately after each use, the HygienicTray must be cleaned, disinfected and sterilized.
- During the sterilisation keep the HygienicTray separate from non-stainless steel instruments, such as rubber polishers and abrasive stones.
- If disinfection by immersion is being used, it is important to use only cleaning/disinfection solutions that provide corrosion protection and strictly observe the concentrations recommended by the manufacturer.
- Sterilization must be performed with validated procedures. If possible, use a single-pulsed or fractionated vacuum autoclave and subvacuum drying.

In the unlikely case of an incident caused by the Tray please refer to the manufacturer and competent authorities.

Intensiv SA declines any liability for any use of the Tray other than that specified in these instructions for use.

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