Intensiv ProxoContour

Pat. 699 819

DE

Produktbeschreibung

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EN

Product description

- Single sided diamond coated metal strip with two working parts of different grit size, 60µm (brown) and 40µm (red)
- Perforated anatomical ends for a better gripping
- Height: 2.5 mm
- Total length: 80 mm
- Thickness: 0.05 mm
- Sterilizable and reusable

Intended use

- Modeling and finishing of fillings in the interproximal area.
- Removal of excess material and overhangs of fillings.

Practical Tips

- Application by dentist only.
- The total length of the strip (80 mm) allows an optimal finger support on the adjacent teeth during work.
- To get through the contact area, use the uncoated part of the strip and exert a slight horizontal movement. Do not twist, bend or pull the strip too tightly during use.
- After introduction, work with the less abrasive side (40µm, red).
- Then apply the 60µm diamond-coated side, with cautious horizontal movements, to model the proximal contours.
- Next, switch again to the less abrasive side (40µm, red) to complete the procedure with the polishing.
- ProxoContour must be removed with a vertical movement after repositioning of the uncoted segment portion under the contact area.
- In the event the contact area is very tight, it is useful to start by inserting a wooden wedge to open up a slight separation between the teeth before placing ProxoContour in the area to be treated.
- After finishing, polish proximal surfaces. We recommend ProxoPolish with 15µm and 8µm diamond grit.

Maintenance and sterilisation

- ProxoContour is supplied non-sterile. Prior to first use on the patient and immediately after each use, the instruments must be cleaned, disinfected and sterilized.
- If disinfection by immersion is being used, it is important to use only cleaning/disinfection solutions that provide corrosion protection and strictly observe the concentrations recommended by the manufacturer.
- Sterilization must be performed with validated procedures. If possible, use a single-pulsed or fractionated vacuum autoclave and subvacuum drying.
- Full cleaning, disinfection, sterilization, checking and maintenance instructions can be requested at info@intensiv.ch or visualized through the QR-code on the bottom of the page.

Risk Warnings

- Tilting or rotating the ProxoContour metal strip during use can lead to fracture and/or fissure.
- Pulling too forcefully in tight interdental spaces or beginning with the more abrasive (60µm) grit can cause the fissures.
- Bending the ProxoContour metal strip causes possible breaking points and can lead to detachment of the diamond coating.
- Leaving proximal surfaces roughened with 40µm grain size can lead to undesirable mechanical retention and formation of plaque.

In the unlikely case of an incident caused by the instrument please refer to the manufacturer and competent authorities.

Intensiv SA declines any liability for any use of the instrument other than that specified in these instructions for use.

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EN

Product description

- Metal strip, with only one diamond-coated side and two working parts in different grain sizes, 15µm (yellow) and 8µm (orange)
- Perforated anatomical ends for a better gripping
- Height: 2.5 mm
- Total length: 80 mm
- Thickness: 0.05 mm
- Sterilizable and reusable

Intended use

- Removal of pigmentations in the interproximal areas.
- Polishing of interproximal areas.

Practical Tips

- Application by dentist only.
- The total length of the strip (80 mm) allows an ideal support of fingers on adjacent teeth during treatment.
- To introduce the strip through the contact area, use the uncoated part and apply a slight horizontal movement. Do not twist, bend or pull strip too tightly during use.
- After introducing the strip, apply a back and forth movement, first with the slightly more abrasive side (15µm, yellow).
- To complete the treatment of surfaces with polishing, finish with less abrasive side (8µm, orange).
- In order to remove ProxoPolish, relocate below the contact area with the uncoated side and pull the strip out vertically.
- In case the contact area is very tight, proceed to preliminary insertion of a wooden wedge to allow a minimum separation between the teeth before placing the strip in the treatment area.

Maintenance and sterilisation

- ProxoPolish is supplied as a non-sterile product. At first use on patient and after each use the instrument must be cleaned, disinfected and sterilized.
- If disinfection by immersion is performed, use only cleaning/ disinfection solutions providing protection against corrosion and adhere closely to the concentrations specified by the manufacturer.
- Sterilization must be carried out in accordance with validated procedures. If possible, use vacuum autoclave with single or fractionated vacuum and vacuum drying.
- Full cleaning, disinfection, sterilization, checking and maintenance instructions can be requested at info@intensiv.ch or visualized through the QR-code on the bottom of the page.

Risk Warnings

- Tilting or rotating the ProxoPolish during use can lead to fractures and/or fissures.
- Bending of ProxoPolish causes possible breaking points and can lead to detachment of the diamond coating.

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