DE

## Intensiv ProxoStrip Anterior Pat. 699 819

### Produktbeschreibung

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## Intensiv ProxoStrip Anterior Pat. 699 819

### Product description

- Metal strip, diamond-coated on one side only with two working parts of different grit size: 40µm (red) and 15µm (yellow)
- Perforated anatomical ends for a better gripping
- Height: 2.5mm
- Total length: 55mm
- Thickness: 0.05mm
- Sterilizable and reusable

### Intended use

- Finishing and polishing of interproximal aesthetic restorations for anterior teeth
- Beveling of the interproximal areas

### Practical Tips

- Application by dentist only.
- The reduced length of the strip allows an optimal finger support on the anterior adjacent teeth.
- To get past the contact area, use the uncoated part of the strip and exert a slight horizontal movement.
- After introduction, work with the less abrasive side (yellow, 15µm), in order to facilitate the access of the more abrasive side (40µm, red). Then begin the finishing procedure using a back-and-forward movement.
- Next, switch again to the less abrasive side (15µm, yellow) to complete the procedure with the polishing.
- ProxoStrip Anterior must be removed with a vertical movement after repositioning it to the uncoated segment.
- In the event the contact area is very tight, it is useful to start by inserting a wooden wedge to open up a slight separation between the teeth before placing ProxoStrip Anterior in the area to be treated.

### Maintenance and sterilisation

- Intensiv ProxoStrip Anterior is supplied non-sterile. Prior to first use on the patient and immediately after each use, the instruments must be cleaned, disinfected and sterilized.
- Keep Intensiv ProxoStrip Anterior separate from non-stainless steel instruments, such as rubber polishers and abrasive stones.
- If disinfection by immersion is being used, it is important to use only cleaning/disinfection solutions that provide corrosion protection and strictly observe the concentrations recommended by the manufacturer.
- Sterilization must be performed with validated procedures. If possible, use a single-pulsed or fractionated vacuum autoclave and subvacuum drying.
- Full cleaning, disinfection, sterilization, checking and maintenance instructions can be requested at [info@intensiv.ch](mailto:info@intensiv.ch) or visualized through the QR-code on the bottom of the page.

In the unlikely case of an incident caused by the instrument please refer to the manufacturer and competent authorities.

Intensiv SA declines any liability for any use of the instrument other than that specified in these instructions for use.

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