

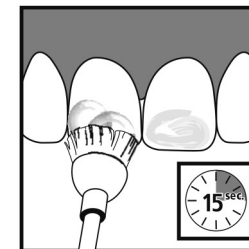
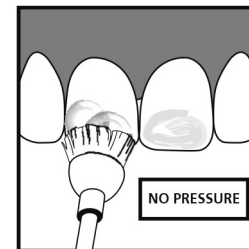
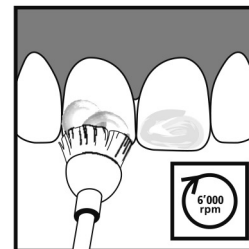
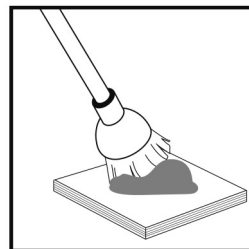
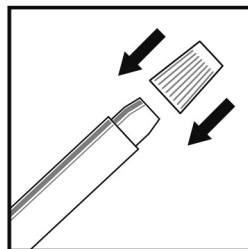
Instructions for use Intensiv UniglossPaste

REF 2500

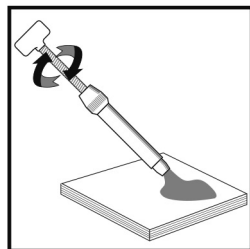IFU-5-10/1/2022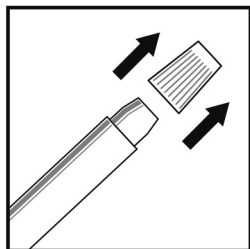

DE

Intensiv UniglossPaste

Produktbeschreibung

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Intensiv UniglossPaste

Description of product

- Universal One-Step extra fine grit diamond paste for natural high gloss shining of composite surfaces in one single application.

Intended use

- Final polishing and high gloss shining for the surfaces of composite restorations.
- Maintenance of existing composite restorations.

Instructions for use

- Application by dentist only.
- Do not use the paste after its shelf life date.
- To avoid paste drying out please ensure the syringe cap is tightly closed after usage.
- The use of natural raw materials may cause a slight change in colour.
- It is recommended that both patient and operator protect their eyes.
- To avoid cross contamination use the product with new instruments separated from the rest of the instrumentation.
- Do not swallow or inhale the paste.
- If not used for a long time, please add a little quantity of water to return to the previous consistency.

In the unlikely case of an incident caused by Intensiv UniglossPaste please refer to the manufacturer and competent authorities.

Intensiv SA declines any liability for any use of Intensiv UniglossPaste other than that specified in these instructions for use.

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